

Please note when registering your test you must provide all the information requested, even if you have already provided some of this information when you ordered your test. The information you give on this form will be included on your Fit to Fly Certificate. Failure to provide full and accurate information will result in delays to your Fit To Fly certificate (if one has been requested).

Scan here to register your
Coronavirus Test by visiting:

[www.blindspotglobal.com/
register-your-coronavirus-test/](http://www.blindspotglobal.com/register-your-coronavirus-test/)






Collect nasal sample

Blow your nose once using a tissue. Insert the swab into one nostril. Slowly slide it approx. 2cm forward. Rotate the swab 4 times (for approx. 15 seconds) against the lining of the nasal wall, before removing it. Repeat in other nostril using the same swab



Prepare Sample

Insert the swab into an extraction buffer tube, squeeze the tube and stir the swab more than ten times. Remove the swab while squeezing the sides of the tube. Press the nozzle cap tightly on the tube

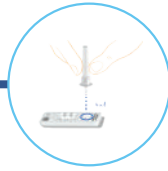

Performing Test


Add 4 drops of extracted sample vertically into the specimen well of the test device.



Performing Test

Add 4 drops of extracted sample vertically into the specimen well of the test device.

RESULTS INTERPRETATION

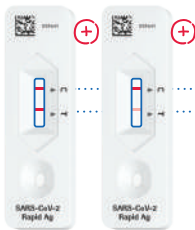

Positive Test Result

If both the colored line C and T are visible, the test result is positive. This means the test detected the virus protein in the sample. The tested person is currently infected with SARS-Cov-2.



Negative Test Result

A visible control line C alone means the test worked correctly. The test result is negative. No virus protein could be detected in the sample

Control Line C:
Assures test is working correctly

Control Line T:
Visible if SARS-CoV-2 antigen was detected



Invalid Test Result

If there is no line visible, or only the line marked with a T, the test did not work correctly and needs to be repeated with another test device.

Scan here to register your
Coronavirus Test by visiting:

[www.blindspotglobal.com/
register-your-coronavirus-test/](http://www.blindspotglobal.com/register-your-coronavirus-test/)

