DE deutsch Gebrauchsanweisung
EN english Instructions for use
FR français Mode d'emploi
ES español Instrucciones de uso

IT italiano Istruzioni per l'uso
NL nederlands Gebruiksaanwijzing
CS český Návod k použití
ET eesti Kasutusjuhend

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EN ActiveColor Sport thumb Splint

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a neoprene support with a thumb splint for stabilising the metacarpophalangeal joint and saddle joint of the thumb.

Indications

Overload prophylaxis, pulled muscle, sprain, load-induced irritation.

Contraindications

Peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

Application risks/Important notes

The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- remove it during longer periods of rest, e. g. when sleeping
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use – this medical aid is intended for treating one patient
- whilst wearing the support, please neither use any creams nor ointments on or around it as they can destroy the material

Fitting instructions

Place the support on the metacarpophalangeal joint affected by guiding the thumb through the intended thumb opening. Remove the Velcro protection on the palm surface and wind the support around the metacarpophalangeal joint. Finally, attach the Velcro to the soft lining intended for this purpose.

Removal

To remove the support, open the Velcro fastener. Pull the support off over the thumb. Close the Velcro fastener in order to avoid early wear to the Velcro surface.

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