

## Instructions for SmartBond SC

Orthodontic adhesive (gel) for metal, plastic and ceramic brackets.  
Bonds to composite and porcelain materials. Ethyl Cyanoacrylate.  
Instructions refer to images in instruction booklet SM-007.

### Bonding instructions

1. Clean
2. Rinse
3. Etch (10-20 seconds)
4. Rinse thoroughly – do not dry
5. Apply adhesive sparingly
6. Align on wet surface; ensure that there is no gap between bracket and tooth
7. Adjust max. 5 seconds and press
8. Bond sets in 3-5 minutes

### Dispensing instructions

1. Twist handle
2. Push handle gently with your palm or thumb to dispense adhesive
3. Pull back handle to stop afterflow; avoid air into syringe
4. If air pocket occurs – carefully push out air with handle
5. Ensure proper closing to avoid drying out



### Warning!

- Irritant
- Bonds skin in seconds

A notice to the user and/or patient that if any serious incident occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

