

5 tips to get the best from your rechargeable Oticon Opn hearing aids

- 1. Charging** Always fully charge the batteries, until the light in the charging dock is solid green - it takes approximately 7 hours.
- 2. Out of power** If the batteries are drained and the hearing aids have switched off, never try to get more use by opening and closing the drawer. Either:
 - a. Place the hearing aids in the charging dock and ensure it is powered on.
 - b. Insert disposable batteries and keep the rechargeable batteries safe for charging later.
- 3. Not in use** If you are not using your hearing aids for an extended period of time, take the batteries out.

Note: Avoid leaving the rechargeable batteries in the hearing aids with the battery drawers open. This will compromise battery life.
- 4. Handling** Don't keep the rechargeable batteries together with metal objects such as keys and coins.
- 5. Maintenance** Wipe off any moisture on the hearing aids or charging dock before charging using a soft tissue. Do not use rubbing alcohol or other chemical substances.

