



PLEASE READ INSTRUCTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SYSTEM DAMAGE OR FAILURE.

WARNING

The floor pressure mat is an aid only and may not be suitable for all patients. This product is not a substitute for staff vigilance or visual monitoring.

SAFETY

Always test mat before each use using the following procedure

1. Step on the floor mat
2. The alert will sound on the monitor or nurse call system
3. Reset the alert

If the alert does not sound

1. Check your connections
2. Test again
3. If it still does not sound, do not use the mat. Contact your distributor for further assistance

CLEANING

Wipe the mat with a clean, damp cloth. A small amount of disinfectant, alcohol based cleaning product or diluted, chlorine based cleaning solution may be used. Do not submerge the mat.

MAT CARE

- ✔ Store Flat. Do not bend, fold or roll the mat as this will void warranty.
- ✔ Do not subject the mat to: Mechanical shock, excessive humidity, extremes of temperature, corrosive liquids.
- ✔ This product is designed for indoor use only.
- ✔ It is recommended to place mat logo side down when in use.
- ✔ The mat should be replaced 1 year from date of first use. Do not use the mat after this date.
- ✔ The mat should not be disposed of with household waste. Please follow the local authority's instructions for disposal of waste electrical products.

LIMITED WARRANTY

This disposable sensor mat is guaranteed against manufacturers defects for a period of one year from date of sale and will be repaired or replaced if returned to the distributor. Defects caused by exposure to fluids, folding, creasing and any other misuse will void the warranty. No other warranty is given or implied.