

# How to pair a LINAK® Wireless FS3 Foot Switch With CHANGE Chairs



1  
Open battery cover on back of FS3. Mount batteries and move within 2 meters of the control box.



2  
Activate Direct Pairing by pressing button under FS3 battery cover for 3 seconds – Buzzer and LED is activated.



3  
Move foot switch within 10 cm of the control box until buzzer frequency change from slow to fast.



4  
Confirm pairing by pressing button under FS3 battery cover.



Beep  
Beep



Double confirmation beep - Pairing OK



How to pair